

Competency Assessment Checklist for Medstrom MMO5000 Bed.



Staff Name:

Location:

Date:

MEDSTROM BED MMO5000	Be aware of and indicate function of the following:	Y	N
1. Identifies anatomy of the MMO5000 Bed	• Safe working load : 250kg Max patient weight: 215kg.		
	• Back Rest		
	• Height Adjustment		
	• Thigh Section		
	• Chair Position		
	• Trendelenburg/Reverse Trendelenburg		
	• Electric CPR		
	• Calf section(manual)		
	• CPR Function (manual)		
	• Lock Out Functions		
2. Operation	Demonstrate how to operate the device safely	Y	N
	1. Demonstrate correct procedure for moving the bed laterally and around the ward		
	2. Demonstrate correct procedure engaging brake and steer		
	3. Demonstrate how to activate electric CPR on patient / carer handset		
	4. Demonstrate how to activate manual CPR function		
	5. Demonstrate how to put bed in cardiac chair position		
	6. Demonstrate how to raise and lower manual calf section		
	7. Demonstrate how to lock out functions on patient / carer handset		
	8. Demonstrate how to set custom height setting on patient / carer handset		
	9. Demonstrate how to sit and mobilise safely from and into the bed		
	10. Demonstrate how to remove head & footboards		
	11. Demonstrate how to remove and fit sleep deck panels		
	12. Demonstrate how to extend the bed frame and platform		
	13. Demonstrate how to lower and raise side rails		

	14. Demonstrate how to remove side rails (3/4 folding only)		
	15. Demonstrate how to use accessories safely (if relevant)		
	16. Deliver training session covering main points indicated for 'competent' level		

MEDSTROM BED MMO5000	Be able to explain the following	Y	N
	17. Explain the benefits of the bed in terms of minimising injuries from falls from bed		
	18. State low and high heights and benefits of high height		
	19. Explain purpose of the Safe Stop 3cm before lowest height		
	20. Explain what happens if brake is off when bed travels to low position		
	21. Explain the benefits of the custom height setting		
	22. Explain the functions of the brake and steer and when to engage/disengage steer		
	23. Explain the controls on patient / carer handset		
	24. Explain the benefits of the Safe Stops on backrest		
	25. Explain the benefit of the lengthening backrest		
	26. Explain how to clean the bed using hospital protocol, locking out height function for safety		
	27. Identify the SWL and MPW of the bed		

Statement – I have demonstrated competence and knowledge relevant to this device. I certify that I'm aware of my professional responsibilities for continuing professional development and I am accountable for my own actions. I am competent to use this product without further training.

Trainee's Signature:

Date:

Competence Verified By:

Name (Print):

Trainer's Signature:

Date: