

**Competency Assessment Checklist for Medstrom MMO6000 Bed.**



**Staff Name:**

**Location:**

**Date:**

MEDSTROM BED MMO6000	Be aware of and indicate function of the following:	Y	N
<b>1. Identifies anatomy of the MMO6000 Bed</b>	<ul style="list-style-type: none"> <li>• Safe working load: 250kg Max patient weight: 215kg.</li> </ul>		
	<ul style="list-style-type: none"> <li>• Back Rest</li> </ul>		
	<ul style="list-style-type: none"> <li>• Height Adjustment</li> </ul>		
	<ul style="list-style-type: none"> <li>• Thigh Section</li> </ul>		
	<ul style="list-style-type: none"> <li>• Chair Position</li> </ul>		
	<ul style="list-style-type: none"> <li>• Progression to front exit position</li> </ul>		
	<ul style="list-style-type: none"> <li>• Trendelenburg/Reverse Trendelenburg</li> </ul>		
	<ul style="list-style-type: none"> <li>• Electric CPR</li> </ul>		
	<ul style="list-style-type: none"> <li>• Calf Section ( manual)</li> </ul>		
	<ul style="list-style-type: none"> <li>• CPR function ( manual)</li> </ul>		
	<ul style="list-style-type: none"> <li>• Lock Out Functions</li> </ul>		
	<ul style="list-style-type: none"> <li>• Brake</li> </ul>		
	<ul style="list-style-type: none"> <li>• Steer lock function to assist with moving the bed</li> </ul>		
	<ul style="list-style-type: none"> <li>• Moving bed laterally</li> </ul>		
<ul style="list-style-type: none"> <li>• Operating side rails</li> </ul>			
<ul style="list-style-type: none"> <li>• Integrated Aerospacer Surface</li> </ul>			
<b>2. Operation</b>	<b>Demonstrate how to operate the device safely</b>	<b>Y</b>	<b>N</b>
	<ul style="list-style-type: none"> <li>• Demonstrate correct procedure for moving the bed laterally and around the ward</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate correct procedure engaging brake and steer</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to activate electric CPR on patient / carer handset</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to activate manual CPR function</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to put bed in cardiac chair position</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to achieve foot egress safely</li> </ul>		

	<ul style="list-style-type: none"> <li>• Demonstrate how to raise and lower manual calf section</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to lock out functions on patient/ carer handset</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to set custom height setting on patient / carer handset</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to remove head &amp; footboards</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to remove and fit sleep deck panels</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to lower and raise side rails</li> </ul>		
	<ul style="list-style-type: none"> <li>• Demonstrate how to use accessories safely (if relevant)</li> </ul>		
<b>MEDSTROM BED MMO6000</b>	<b>Be aware of and indicate function of the following:</b>	<b>Y</b>	<b>N</b>
	<ul style="list-style-type: none"> <li>• Deliver training session covering main points indicated for 'competent' level</li> </ul>		
	<b>Be able to explain the following</b>		
	<ul style="list-style-type: none"> <li>• Explain the benefits of the bed in terms of promoting early mobilisation</li> </ul>		
	<ul style="list-style-type: none"> <li>• Explain the patient conditions for whom the bed would be suitable</li> </ul>		
	<ul style="list-style-type: none"> <li>• Explain the benefits of the custom height setting for sitting practice etc.</li> </ul>		
<b>MEDSTROM BED MMO6000</b>	<b>Be aware of and indicate function of the following:</b>	<b>Y</b>	<b>N</b>
	<ul style="list-style-type: none"> <li>• Explain the functions of the brake and steer and when to engage/disengage steer</li> </ul>		
	<ul style="list-style-type: none"> <li>• Explain the controls on patient / carer handset</li> </ul>		
	<ul style="list-style-type: none"> <li>• Explain the benefits of Safe Stops on backrest</li> </ul>		
	<ul style="list-style-type: none"> <li>• Explain the benefits of lengthening backrest</li> </ul>		
	<ul style="list-style-type: none"> <li>• Explain how to clean the bed using hospital protocol, locking out height function for safety</li> </ul>		
	<ul style="list-style-type: none"> <li>• Identify the SWL and MPW of the bed</li> </ul>		

**Statement** – I have demonstrated competence and knowledge relevant to this device. I certify that I'm aware of my professional responsibilities for continuing professional development and I am accountable for my own actions. I am competent to use this product without further training.

*Trainee's Signature:*

*Date:*

**Competence Verified By:**

*Name (Print):*

*Trainer's Signature:*

*Date:*