

Competency Assessment Checklist for Medstrom MMO8000 Bed.



Staff Name:

Location:

Date:

MEDSTROM BED MMO8000	Be aware of and indicate function of the following:	Y	N
1. Identifies anatomy of the MMO6000 Bed	• Safe working load: 340kg Max patient weight: 250kg.		
	• Back Rest		
	• Height Adjustment		
	• Thigh Section		
	• Chair Position		
	• Trendelenburg/Reverse Trendelenburg		
	• Electric CPR		
	• Calf Section (manual)		
	• CPR function (manual)		
	• Lock Out Functions		
	• Brake		
	• Steer lock function to assist with moving the bed		
	• Moving bed laterally		
	• Operating side rails		
• Head and foot boards			
• Operating bed extension			
2. Operation	Demonstrate how to operate the device safely	Y	N
	• Demonstrate correct procedure for moving the bed laterally and around the ward		
	• Demonstrate correct procedure engaging brake and steer		
	• Demonstrate how to activate electric CPR on patient / carer handset		
	• Demonstrate how to activate manual CPR function		
	• Demonstrate how to put bed in cardiac chair position		

	<ul style="list-style-type: none"> • Demonstrate how to achieve foot egress safely • Demonstrate how to raise and lower manual calf section 		
	<ul style="list-style-type: none"> • Demonstrate how to lock out functions on patient/ carer handset • Demonstrate how to set custom height setting on patient / carer handset • Demonstrate how to remove head & footboards • Demonstrate how to remove and fit sleep deck panels • Demonstrate how to lower and raise side rails • Demonstrate how to use accessories safely (if relevant) 		
MEDSTROM BED MMO 8000	Be aware of and indicate function of the following:	Y	N
	<ul style="list-style-type: none"> • Deliver training session covering main points indicated for 'competent' level 		
	Be able to explain the following		
	<ul style="list-style-type: none"> • Explain the benefits of the bed in terms of promoting early mobilisation • Explain the patient conditions for whom the bed would be suitable 		
	<ul style="list-style-type: none"> • Explain the benefits of the custom height setting for sitting practice etc. 		
MEDSTROM BED MMO 8000	Be aware of and indicate function of the following:	Y	N
	<ul style="list-style-type: none"> • Explain the functions of the brake and steer and when to engage/disengage steer • Explain the controls on patient / carer handset • Explain the benefits of Safe Stops on backrest • Explain the benefits of lengthening backrest • Explain how to clean the bed using hospital protocol, locking out height function for safety • Identify the SWL and MPW of the bed 		

Statement – I have demonstrated competence and knowledge relevant to this device. I certify that I'm aware of my professional responsibilities for continuing professional development and I am accountable for my own actions. I am competent to use this product without further training.

Trainee's Signature:

Date:

Competence Verified By:

Name (Print):

Trainer's Signature:

Date: