

IMPORTANT SAFETY INFORMATION: DO NOT REMOVE
Quick User Guide: 5000 ultra-low bed


Safe working load: 250kg Max patient weight 215kg

BED OPERATION: HAND CONTROL

1. BACKREST Press the left side of the button to raise the backrest or the right side of the button to lower. It will pause at 30° and 45°.

2. HEIGHT ADJUSTMENT Press the left side of the button to raise or the right side of the button to lower. When the right-hand side of the height adjustment button is pressed and held down the bed will stop 3cm before it reaches its lowest position of 21cm. For safety reasons the bed will sound an alarm as it travels to the lowest position. **PLEASE NOTE:** The bed brake and steer pedals automatically engage when the bed reaches its lowest position. The bed must be raised by 5cm in order to manually release the brake and move the bed.

A custom height position can be set by pressing and holding both chair position buttons (4) for at least 10s until a beep is heard.

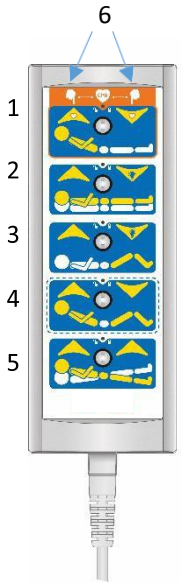

3. THIGH SECTION
Press the left side of the button to raise or the right side of the button to lower. Make sure the calf section is in the flat position before lowering the thigh section.

4. CHAIR POSITION Press the left side of the button to achieve a full chair position. This combines back and thigh adjustment with reverse Trendelenburg. Press the right side to reverse back to flat. **PLEASE NOTE: When the bed is in the ultra-low position, pressing the left-hand chair position button will cause the bed to rise before it moves into chair position.**

5. TRENDELENBURG/REVERSE TRENDELENBURG Can be operated with the bed in flat or chair position. Press the right side of the button to achieve the Trendelenburg position until the bed reaches the angle desired. To put back to horizontal, press the left side of the button. When the bed reaches the horizontal position, a 1 second pause happens. If you continue pressing the button, the Reverse Trendelenburg movement begins. Note: The Trendelenburg/Reverse Trendelenburg position is related to clinical indications and the function should **only** be used by the carer. Ensure the bed is at least 30cm from the wall before operating. To avoid misuse the function should be disabled when not required by using the lock-out function described below.

6. ELECTRIC CPR Press and hold both backrest buttons together to return the bed to flat position and an appropriate height for cardiac massage.

ADDITIONAL FEATURES

CALF SECTION (MANUAL):

The adjustment is made using a 4-position ratchet mechanism. Pull the end of the calf section to raise. To lower, raise fully then lower.

CPR FUNCTION (MANUAL): In case of medical emergency pull the orange handle on either side of the backrest. This allows the backrest to move to the CPR position quickly. For safety reasons, the lowering of the backrest in this way is dampened.

For support 24/7/365 UK 0845 371 1717 IRELAND 01 686 9487

DOC477_5000 QUG_REV 1 June19

IMPORTANT SAFETY INFORMATION: DO NOT REMOVE

Quick User Guide: 5000 ultra-low bed

LOCK OUT FUNCTIONS: The lock out facility is located between the buttons of each function and is operated by using the key attached to the hand control cable. To operate the lock out function locate the pins of the key into the two recesses on the feature and rotate to the right.

OPERATING THE BRAKE

The brake pedal is at the foot of the bed. Ensure the brake is applied at all times. To release, raise the lever. The bed brake automatically engages when the bed reaches its lowest position. The bed must be raised by 5cm in order to manually release the brakes and move the bed.

USING THE STEER LOCK FUNCTION TO ASSIST WITH MOVING THE BED

Manufacturer's guidance is to push from the foot of the bed.

The steer pedal is at the head end. This should be engaged (in the 'down' position) at all times when the bed is stationary or being manoeuvred as it will allow controlled movement without effort, significantly reducing the force required to move the bed.

Only disengage the steer pedal by pushing up with your foot if you require lateral (sideways) movement of the bed. Re-engage the steer pedal by pressing down on the pedal when travelling straight in a corridor or leaving the bed unattended. To replace the bed into a bed space, make sure steer is engaged, push the bed straight into place and engage the brake.

MOVING THE BED Laterally

If the bed needs to be moved laterally the steer lock at the head end should be released (up position). The steer lock should be re-engaged when the bed has been repositioned.

OPERATING THE SIDE RAILS

¾ Folding Side Rails: To lower, face towards the foot end of the bed, place the hand nearest the side rail on the top rail and the other hand on the handle of the second rail (shown right). Gently pull the handle in the direction of the orange arrow. As soon as the handle moves, use your other hand to start to lower the side rail towards the foot of the bed and let go of the handle.



To raise, grab the top bar and pull it up and toward the head of the bed. There will be a click when it is locked into position.

Split side Rails: Can be released by squeezing the grey handle. The side rail will lower with dampened action. To raise simply lift until the side rail locks into position with a 'click'.



OPERATING THE BED EXTENSION

The bed incorporates a length extension of 18cm as standard. To operate, pull the lever at the foot end of the bed (patient left) and pull out the extender from the base of the footboard.

The mattress platform can also be extended by lifting off the sleep deck panel and pulling the bottom of the frame towards the footboard. The panel should then be replaced, ensuring it is firmly clicked into place.