

Quick User Guide: MMO 8000

Max safe working load: 340kg


Max patient weight 250kg

BED OPERATION: HAND CONTROL

- 1. BACKREST** Press the left side of the button to raise the backrest or the right side of the button to lower. It will pause at 30° and 45°.
- 2. HEIGHT ADJUSTMENT** Press the left side of the button to raise or the right side of the button to lower. When the right hand side of the height adjustment button is pressed and held down the bed will stop 3cm before it reaches its lowest position of 21cm then it will sound an alarm as it travels to its lowest position.

The bed brakes automatically when it reaches its lowest position. The bed must be raised by 5cm in order to manually release the brakes and move the bed.

A custom height position can be set by pressing and holding both chair position buttons (4) for at least 10s until a beep is heard.

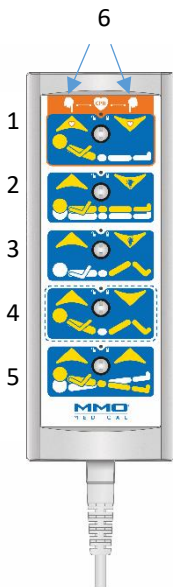

3. THIGH SECTION Press the left side of the button to raise or the right side of the button to lower. Make sure the calf section is in the flat position before lowering the thigh section.

4. CHAIR POSITION Press the left side of the button to achieve a full chair position. This combines back and thigh adjustment with reverse Trendelenburg. Press the right side to reverse back to flat. NOTE: when the bed is in ultra-low position it will rise slightly before moving into chair position.

5. TRENDELENBURG/ REVERSE TRENDELENBURG Can be operated with the bed in flat or chair position. Press the right button of the Trendelenburg function until the bed reaches the angle desired. To put back to horizontal, press the left button of the Trendelenburg function. When the bed reaches the horizontal position, a 1 second pause happens. If you continue pressing the button, the opposite Trendelenburg movement begins. The Trendelenburg position is related to clinical indications and should be used by carer. Ensure the bed is at least 30cm from the wall before operating.

6. ELECTRIC CPR Press and hold both backrest buttons together to return the bed to flat position and an appropriate height for cardiac massage.

ADDITIONAL FEATURES

CALF SECTION (MANUAL):

The adjustment is made using a 6 position ratchet mechanism. Pull the end of the calf section to raise. To lower, raise fully then lower it.

CPR FUNCTION (MANUAL):

In case of medical emergency, pull the red handle on either side of the backrest. This allows the backrest to come to the CPR position quickly. For safety reasons, the lowering of the backrest in this way is dampened.

FOR SUPPORT 24/7/365 CALL:

UK: 0845 371 1717

IRELAND: 01 686 9487

Quick User Guide: MMO 8000

LOCK-OUT FUNCTIONS: Each function may be locked out. Turn the pins using the key provided on the cable to the closed lock position to lock-out a function. To release the function, turn to unlock position.



OPERATING THE BRAKE

The brake pedal is at the foot of the bed. Ensure the brake is applied at all times. The brakes will engage automatically when the bed is in the ultra-low position. To release, raise the lever.

USING THE STEER LOCK FUNCTION TO ASSIST WITH MOVING THE BED

The steer lock is at the head end. This should be in the 'down' position at all times when the bed is stationary or being manoeuvred. The steer lock assists in the easy steering of the bed, significantly reducing the force required.

MOVING THE BED LATERALLY

If the bed needs to be moved laterally the steer lock at the head end should be released (up position). The steer lock should be re-engaged when the bed has been repositioned.

OPERATING THE SIDE RAILS

To lower, face towards the foot end of the bed, place the hand nearest the side rail on the top rail and the other hand on the handle of the second rail (shown right).

Gently pull the handle in the direction of the orange arrow. As soon as the handle moves, use your other hand to start to lower the side rail and let go of the handle.



To raise, grab the top bar and pull it up and toward the head of the bed. There will be a click when it is locked into position.

HEAD AND FOOT BOARDS

To remove the head and foot boards pull upwards to detach the metal bolt from the locking mechanism. To attach again slot the bolt back in to the locking mechanism and push downwards.

OPERATING THE BED EXTENSION

The bed incorporates a length extension of 18cm as standard. To operate, pull the orange lever at the foot end of the bed and pull out the extender from the base of the footboard.

The mattress platform can also be extended by lifting off the sleep deck panel and pulling the bottom of the frame towards the footboard. The panel can then be replaced, ensuring it is firmly clicked into place.

NOTE: IF THE MMO 8000 IS TO BE USED WITH TURNCAIR 1000 IT MAY BE NECESSARY TO DEFLATE THE SIDE BOLSTERS TO OPERATE THE SIDE RAIL. THE BOLSTERS SHOULD BE REINFLATED DURING A TURN CYCLE OR PRIOR TO TRANSPORT.

**FOR SUPPORT 24/7/365 CALL:
UK: 0845 371 1717 IRELAND: 01 686 9487**